
DYNAMICC




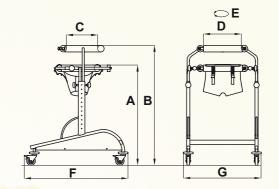

for indoor use

Size	Max. load	Α	В	С	D	E	F	G	Weight of the aid	Turning width
1	kg 55	cm 44-60	cm 52-70	cm 18	cm 25	cm 65	cm 79	cm 66	kg 12	cm 100
2	kg 70	cm 66-82	cm 77-95	cm 21	cm 28	cm 75	cm 84	cm 66	kg 14,6	cm 104
3	kg 8 <mark>5</mark>	cm 77-102	cm 89-116	cm 25	cm 30	cm 85	cm 88	cm 66	kg 15,6	cm 107
4	kg 100	cm 85-112	cm 92-126	cm 30	cm 35	cm 110	cm 95	cm 71	kg 17	cm 117
5	kg 115	cm 98-120	cm 108-138	cm 32	cm 41	cm 120	cm 106	cm 77	kg 19,7	cm 127

Hypen-separated numbers indicate the minimum and maximum range of adjustment

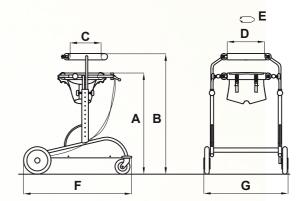

for **outdoor** use

Size	Max. load	Α	В	С	D	E	F	G	Weight of the aid	Turning width
2	kg 70	cm 74-90	cm 85-103	cm 21	cm 28	cm 75	cm 90	cm 74	kg 17,9	cm 110
3	kg 85	cm 85-110	cm 97-124	cm 25	cm 30	cm 85	cm 94	cm 74	kg 19,3	cm 116
4	kg 100	cm 93-120	cm 100-134	cm 30	cm 35	cm 110	cm 101	cm 78	kg 20,4	cm 122
5	kg 115	cm 106-128	cm 116-141	cm 32	cm 41	cm 120	cm 111	cm 82	kg 22,7	cm 135

Hypen-separated numbers indicate the minimum and maximum range of adjustment

CE



Authorized Deal









COMPANY WITH QUALITY
MANAGEMENT SYSTEM
CERTIFIED BY
= ISO 9001=

technical SPECIFICATIONS

for indoor use

This version is available in 5 sizes and allows the complete fitness to each user


Handwheels for the adjustment in height of the axellary and pelvic support.

Non-toxic epoxy varnished chromium-plated frame.

Pelvic harness supporting the user in case of fall. It's adjustable in height thanks to some belts and in girth with velcro strips. It can be easily removed for washing.

Leg divider, holes for its assembling.

Castor wheels. They are all provided with indipendent brakes. The accessory 811 on the front wheels helps the adjustment of flowability of the walking frame, while the accessory 812 on back wheels helps its direction.

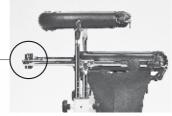

Adjustable handle-bar (813) front holes for its assembling.

Axellary padded support for the trunk, adjustable in height. It can be removed for people with sufficient trunk control.

References for the gradual adjustment in height of the axellary and pelvic support.

Pelvic paded support for the pelvis. It'sadjustable either in height or in front-back direction in order to define the walking about with the moving of the barycentre forwards. It allows an easy access thanks to back opening.

Semi-elastic bumpers for the protection against any bumps.

Weight bars (810), **holes** for their assembling.

for **outdoor** use

This version is **available in 4 sizes** and allows the use of the walking frame on streets, gardens, etc. without any step and steep slope.



17 cm, front pivoting wheels



Hand brakes



30 cm, back fixed wheels

accessories




809 - Arm Supports alternative to 813 Handlebar and 824 Tray (for INDOOR/OUTDOOR use)



(only for INDOOR use)



814 - Leg divider (only for INDOOR use)

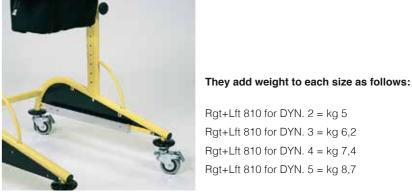

810 - Weight bars (for INDOOR/OUTDOOR use sizes 2 - 3 - 4 and 5)



811 - Front wheels with adjustable brakes 812 - Directional locks for back wheels (only for INDOOR use)



815 - Abduction saddle (only for INDOOR use, sizes 2 - 3 and 4)



813 - Adjustable handlebar alternative to 809-Arm support and 824-Tray (for INDOOR/OUTDOOR use)



824 - Transparent tray alternative to 809-Arm support and 813-Handlebar (only for INDOOR use)







